

COMPLETE ECMO
SOLUTION FOR ADULT,
PAEDIATRIC AND INFANT
PATIENTS

EC[♡]O



14 DAY ECMO MODULES AND TAILORED TUBING SET

ECMO Extra Corporeal Membrane Oxygenation is an extracorporeal technique providing both cardiac and respiratory oxygen support. This technique has proved to be effective in reducing mortality in patients suffering from potentially reversible heart failure and/or severe acute respiratory failure who did not respond to mechanical ventilation with maximal medical and pharmacological treatment⁶.

Eurosets offers physicians a complete solution for adult, pediatric and newborn ECMO patients including PC (PHOSPHORYLCHOLINE) coated oxygenator modules and tailored tubing set, all validated for 14 days of use.

The A.L.ONE ECMO oxygenator family is clinically proven to be an **efficient solution for prolonged cardiorespiratory support**^{1,2,6} and is characterized by an **excellent profile in terms of biocompatibility**³ since the PC coating has been shown to **significantly reduce platelet consumption**⁴.

Moreover, since it is **heparin free**, the A.L.ONE ECMO oxygenator is **suitable for patients with acute HIT** (Heparin-Induced Thrombocytopenia)⁵, where the removal of all sources of heparin is challenging³.




PHOSPHORYLCHOLINE COATING

Phosphorylcholine or PC is the predominant lipid headgroup found within the outer layer of cell membranes.

PC has a natural affinity for water and binds water tightly around itself.

As a result, the outer layer of the cell membrane does not promote clot formation (thrombosis).




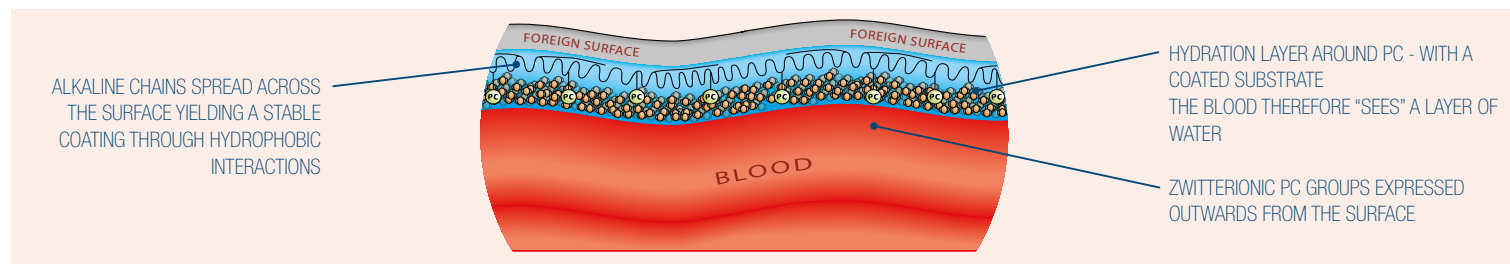


ADULT OXYGENATOR MODULE

TECHNICAL CHARACTERISTICS

PMP fibres	
Priming volume:	225 ml
Contact surface area:	1.81 m ²
Blood flow rate:	0.4 - 7.0 l/min
Heat Exchanger surface area:	0.08 m ²
H.E. Performance Factor:	$\eta = 0.64$ (at 4 l/min)
Material:	Stainless steel
Coating:	Phosphorylcholine (PC)
Venous inlet:	3/8" (9.53 mm)
Arterial outlet:	3/8" (9.53 mm)

14
days

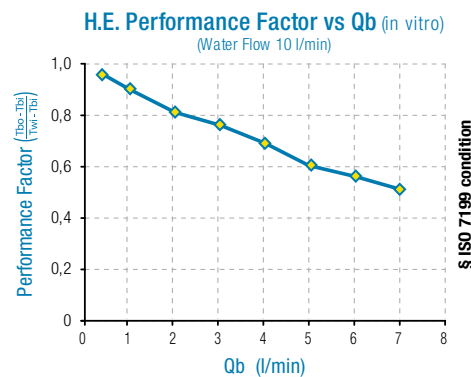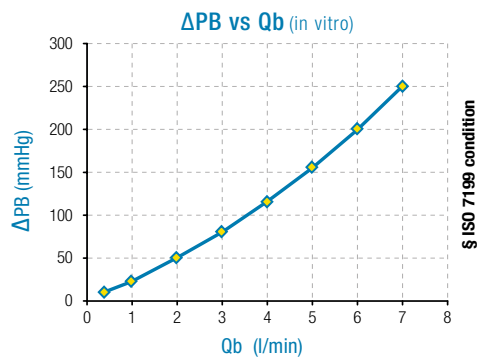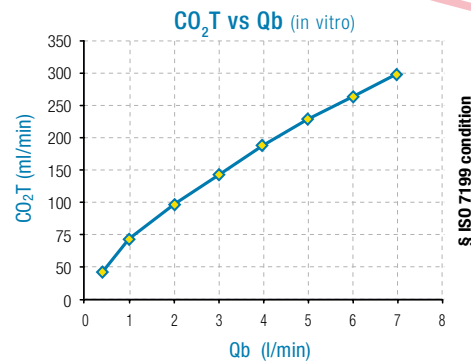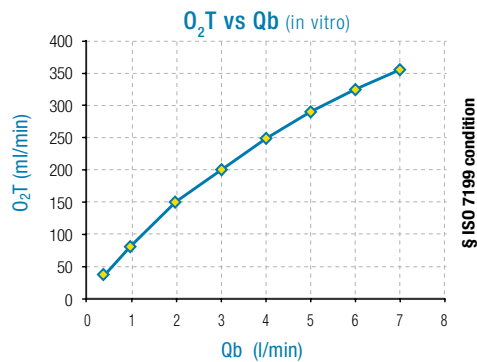





PAEDIATRIC OXYGENATOR MODULE

TECHNICAL CHARACTERISTICS

PMP fibres	
Priming volume:	190 ml
Contact surface area:	1.35 m ²
Blood flow rate:	0.3 - 4.0 l/min
Heat Exchanger surface area:	0.08 m ²
H.E. Performance Factor:	$\eta = 0.64$ (at 4 l/min)
Material:	Stainless steel
Coating:	Phosphorylcholine (PC)
Venous inlet:	3/8" (9.53 mm)
Arterial outlet:	3/8" (9.53 mm)







14
days

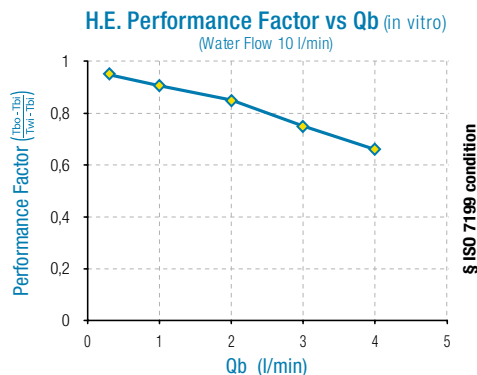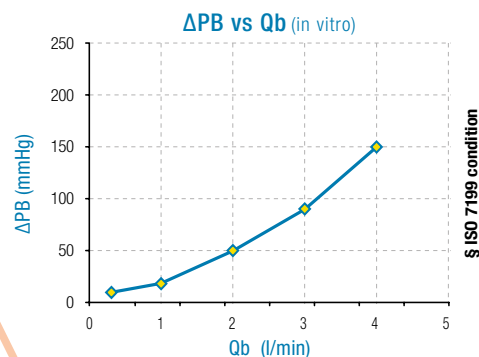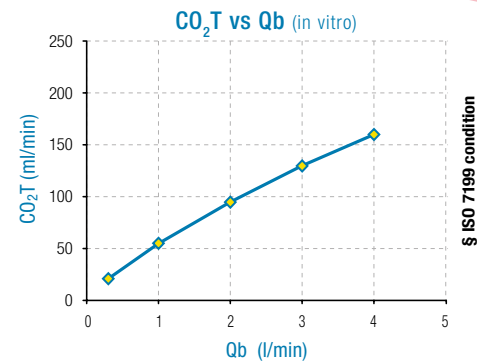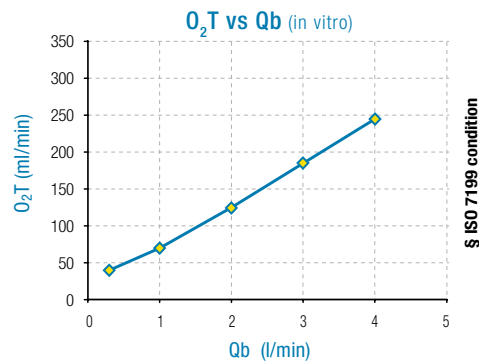

INFANT OXYGENATOR MODULE

TECHNICAL CHARACTERISTICS

PMP fibres	
Priming volume:	90 ml
Contact surface area:	0.69 m ²
Blood flow rate:	0.2 - 1.5 l/min
Heat Exchanger surface area:	0.04 m ²
H.E. Performance Factor:	$\eta = 0.77$ (at 1.5 l/min)
Material:	Stainless steel
Coating:	Phosphorylcholine (PC)
Venous inlet:	1/4" (6.35 mm)
Arterial outlet:	1/4" (6.35 mm)

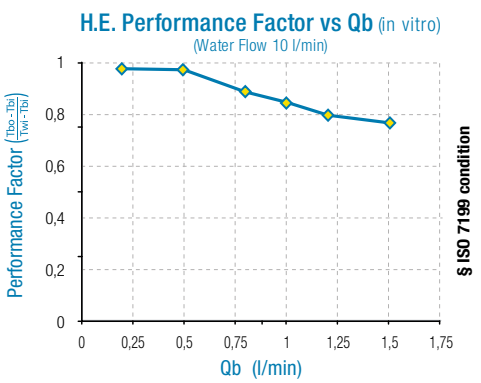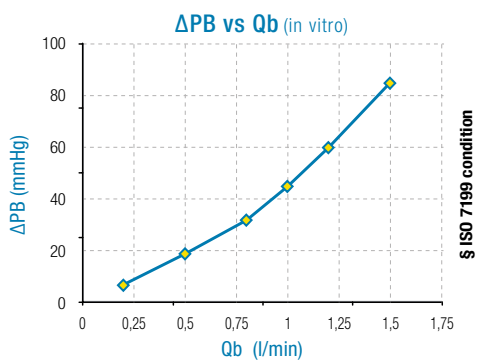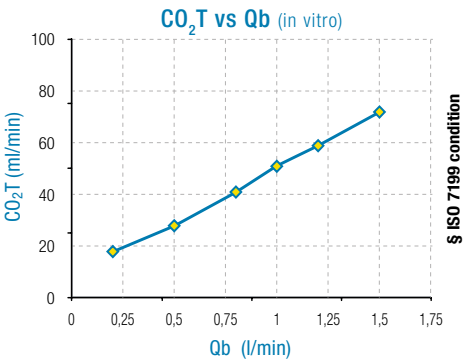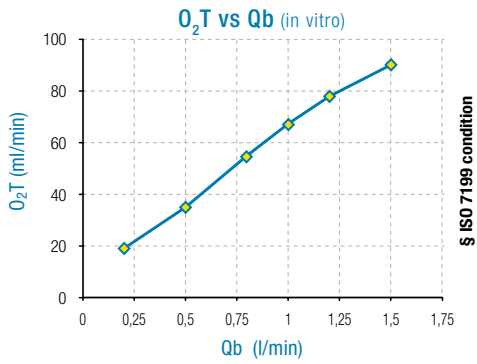

14
days

§ ISO 7199 condition			
Bovine blood	SVO ₂ = 65 ± 5%	B.E. = 0 ± 5 mEq/L	Q _b /Q _g = 1
Hb = 12 ± 1 g/dL	PVCO ₂ = 45 ± 5 mmHg	Temp. = 37 ± 1 °C	FI _{O2} = 100%

ORDERING GUIDE

CODE	DESCRIPTION	NO./PACK
EU5019	ADULT A.L.ONE ECMO 14 days (Totally Coated)	2
EU5045	PAEDIATRIC A.L.ONE ECMO 14 days (Totally Coated)	2
EU5056	INFANT A.L.ONE ECMO 14 days (Totally Coated)	2
EU2257	HOLDER FOR A.L.ONE ECMO	1
EU2498	TOP FIXING HOLDER	1

REFERENCES

1. S.Calvi et al. **55 days ECMO support for post-cardiotomy, by-ventricular failure: a case report.** Abstract presented at 14th FECECT congress, 2011.
2. Nagi el Shouki, M.Stiller. **Case report of 19 days extra corporeal life support with a new ECMO oxygenator.** Abstract presented at 14th FECECT congress, 2011.
3. M.Pieri et al. **A new phosphorylcholine-coated polymethylpentene oxygenator for extracorporeal membrane oxygenation: a preliminary experience.** Perfusion 2012; Volume: 28 issue: 2, page(s): 132-137.
4. R. Lorusso et al. **Effects of phosphorylcholine coating on extracorporeal circulation management and postoperative outcome: a double-blind randomized study.** Interactive CardioVascular and Thoracic Surgery 8 (2009) 7–11.
5. F.Pappalardo et al. **Bioline® heparin-coated ECMO with bivalirudin anticoagulation in a patient with acute heparin-induced thrombocytopenia: the immune reaction appeared to continue unabated.** Perfusion 2009; 24: 135–137
6. M.Belliato et al. **A brief clinical case report of real-time monitoring of oxygenator performances and patient-machine interdependency during a prolonged VvECMO support.** J Clin Monit Comput. 2017 Oct;31(5):1027-1033

EUROSETS EUROPE

ITALY

Eurosets S.r.l.
Strada Statale 12, 143
41036 Medolla (MO)
Italy
Ph: +39 0535 660311
Fax: +39 0535 51248
info@eurosets.com

BENELUX

Eurosets Benelux
Winstar Park,
Rue Provinciale, 62
1301 Bierges (Wavre)
Belgium
Ph: +32 10237993
info.nl@eurosets.com
info.be@eurosets.com

FRANCE

Eurosets France Sarl
Immeuble LE FONTENOT
96, Boulevard Vivier
Merle
69423 LYON Cedex 03
Ph: +33 962191542
Fax: +33 970066573
info.fr@eurosets.com

GERMANY

Eurosets GmbH
Industriestr. 29
82194 Gröbenzell
Ph: +49 81426671227
Fax: +49 81426671229
info.de@eurosets.com

EUROSETS ASIA

CHINA

Eurosets (Suzhou) Medical Device Ltd.
Suite 1308-2, The Summit (South Block)
No. 119, Suzhou Avenue West,
Suzhou Industrial Park - 215000, Greater China.
苏州工业园区苏州大道西119
号苏悦广场南楼1308-2, 215000,
更大的中国
info.cn@eurosets.com

   [eurosets.com](https://www.eurosets.com)

STERILE/EO



CE 0129

Latex free



Prodotto da
Manufactured by
Fabriqué par
Hersteller
Fabricado por

Eurosets s.r.l.
Strada Statale 12, n°143
41036 Medolla (MO), Italy
Tel: +39 0535 660311
Fax: +39 0535 51248
E-mail: info@eurosets.com
eurosets.com

Distribuito da
Distributed by
Distribué par
Vertrieb
Distribuido por